

Topi-CLICK[®] MICRO[™]

Veterinary Patient Instructions




Topical Micro-dosing Application

- ✓ Automated-metered topical dosing provides an accurate and consistent dose every time
- ✓ Audible, Tactile and Visual CLICKs indicate measure of dose (no need to manually mark dose)
- ✓ Innovative TRIAD shape allows for precision handling
- ✓ Built-in REFILL Badge helps pet owners reorder medication on-time



Easy
Screw
Cap

STEP 1:
Unscrew the cap of the Topi-CLICK[®] Micro[™] and set aside for recapping after use.

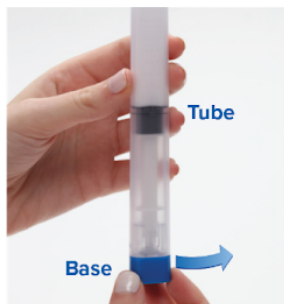

Tube
Base

STEP 2:
Holding the Micro[™] in an upright position, turn/CLICK the **Base** the number of CLICKs or mLs specified on the prescription label. Use the Audible, Tactile, and Visual CLICK points of the triad shaped **Base** as dosing indicators. Be sure to align **Base** with the **Triad Tube** to ensure dose is fully dispensed.



STEP 3:
Apply topical medication with built-in applicator as directed by veterinarian or pharmacist. **Continue rubbing until preparation has completely saturated the prescribed areas through your pet's fur.** **IMPORTANT:** The built-in applicator lessens the risk of transference to unintended areas. No need to use your fingertips to apply medication.



Screw-on

STEP 4:
Screw-on cap after each dose to prevent evaporation. Be sure cap is secured completely.

Innovative
TRIAD Shape
Allows Precision
Application
Control

REFILL Badge[™]



REFILL
Line

Plunger

The Refill Badge[™] reminder system helps patients reorder in a timely manner.

When the **TOP** black line of the **Plunger** reaches the embossed line below the word "REFILL" approximately 32 CLICKs are left. Approximately 12 CLICKs remain at the line above the word "REFILL".

Note: Follow instructions on prescription label before use. Topi-CLICK[®] Micro[™] is not re-fillable.

Made of FDA grade materials | Phthalate free | UV block and is Recyclable ♻️.

